




Quick Start Guide

Series4H

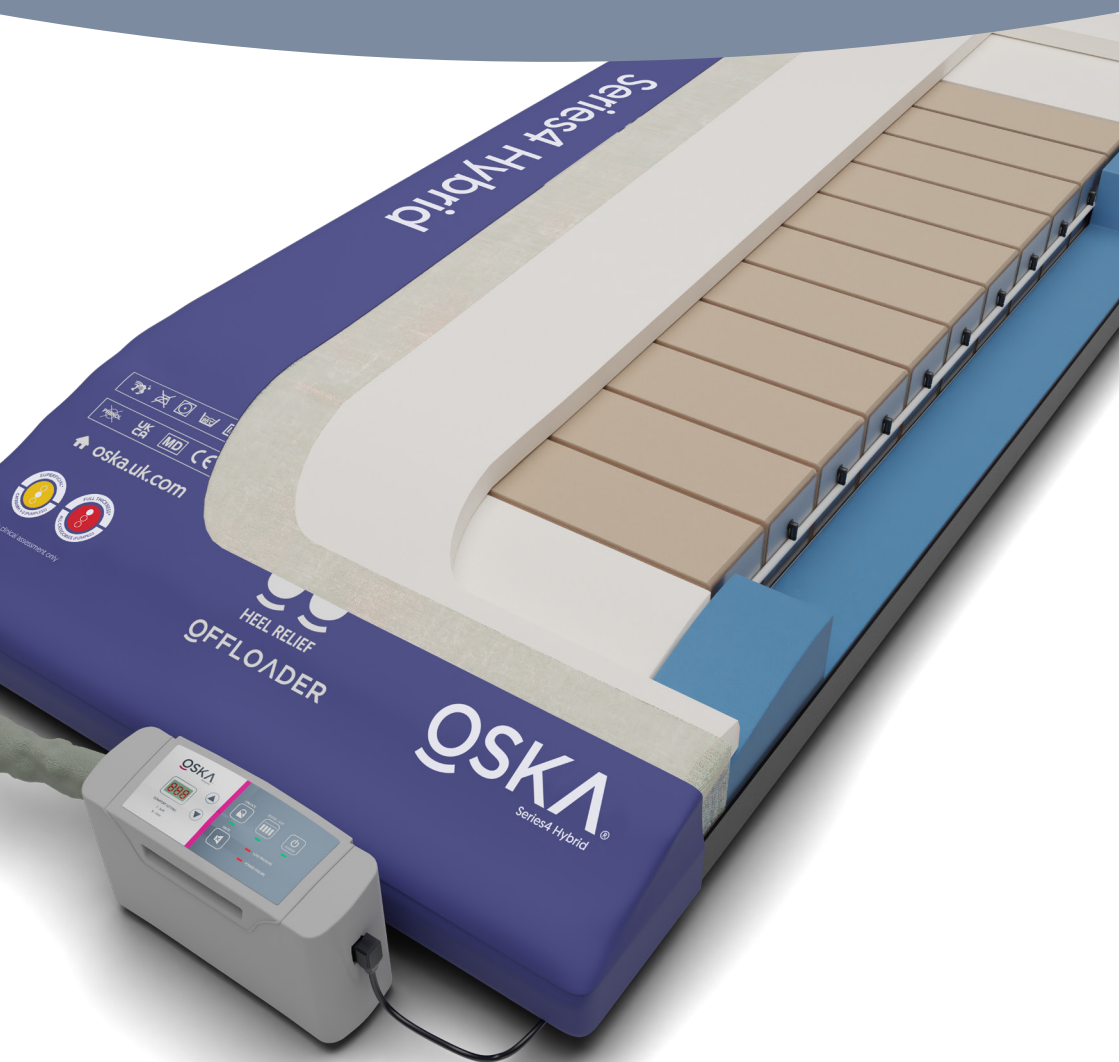

Series4H

Quick start set up guide

For more detailed set up instructions see page 14 of the OSKA Series4H user manual.

1

Laying the mattress on the bed

Place the Series4H mattress on the bed frame with the airline connectors at the foot end of the bed. The blue top cover should face upwards, with the foot icon at the bottom of the bed frame.

The surface is now ready for use in the non-powered mode. That is, without the powered control unit attached.



2

Connecting the pump

Hang the control unit on the foot board at the end of the bed using fold-out hangers. Feed the power cable through the cable tidy on the side of the mattress, plug into the wall & connect to the pump.




3

Connecting airlines to mattress

Align the plastic connector on the end of the umbilical with the 'CPR' connector on the pump.

Push the two fittings together until you hear a 'CLICK'.

Once the mattress and pump have been installed correctly and connected, the system is ready for use.

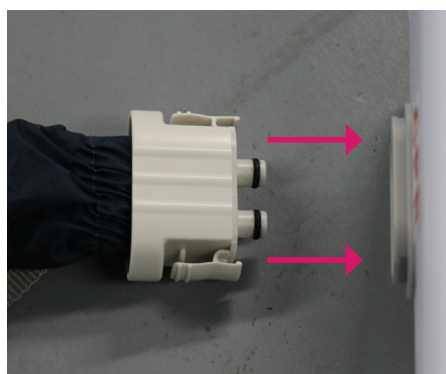

NEED ASSISTANCE? Call 02394 318 318 or email ask@oska.uk.com.

4

Inflating the mattress

Select the appropriate comfort setting for the patient from 1 (Soft) to 8 (Firm)

As the system is inflating, the green 'POWER' indicator light will flash.

Once the system reaches the correct pressure, the flashing green 'POWER' indicator light will automatically be replaced by a constant green indicator light.

The system is now ready for use as a powered hybrid.



5


Static mode

The STATIC mode button can be used to create a static support surface without alternating pressure therapy.

With the pump unlocked, push the STATIC button once to select static mode. The button will click, and you will hear a beep. The green indicator light immediately below the STATIC button will illuminate.

20 minutes after STATIC mode has been selected, the system will automatically return to alternating therapy and the green light below the STATIC button will go out.

To return to an alternating support surface before the default 20 minutes, unlock the control panel and push the STATIC mode button again. The green light will turn off, and the system will return to its alternating therapy.

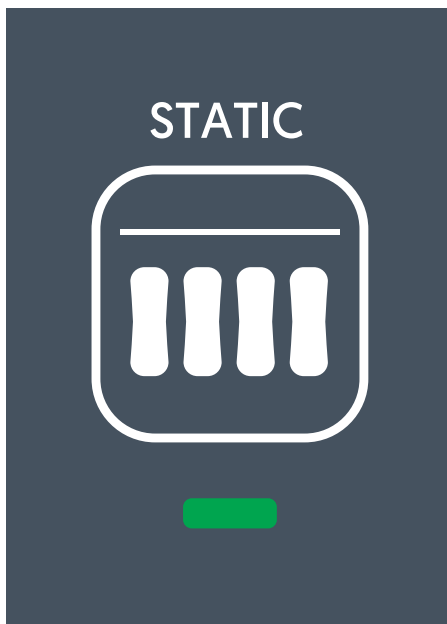

For more detailed set up instructions see page 14 of the OSKA Series4H user manual.

